



Laundering and Care

1. Remove all panels and/ or any metal components prior to cleaning.
2. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
3. Outer shell may be wiped with mild soap and water
4. Do not use any substance containing Chlorine

Warning

1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This brace will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.

Caution

1. Single patient use only
2. Do not over-tighten closures
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.

EUROPEAN AUTHORIZED REPRESENTATIVE



Wellkang Ltd
Suite B, 29 Harley Street,
LONDON, W1G 9QR, U.K.
www.CE-Marking.EU



Council Directive
93/42/EEC of
14 June 1993
concerning Medical Device

MANUFACTURED BY

Dicarre LLC

info@dicarre.com

Toll Free 1-844-DICARRE

125 Half Mile Road, Suite 200
Red Bank, NJ 07701

www.dicarre.com




Dicarre Ankle-Foot
Orthosis, Flat Forefoot
with Liner

DA12

Dicarre Plantar Fasciitis
Night Splint

DA13

User Guide
Application Instruction

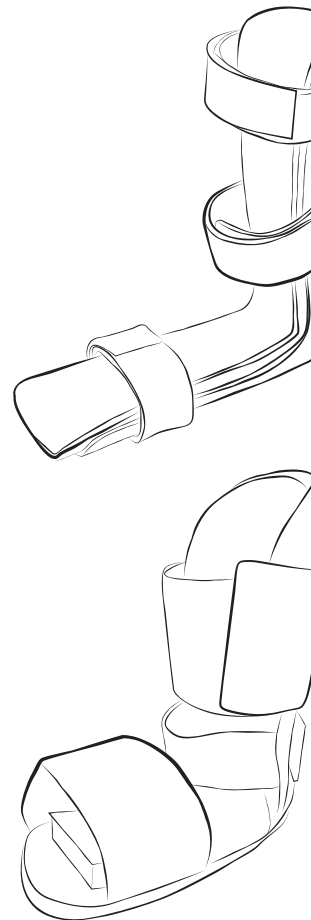

1. Unfasten each of the three straps
2. Position the foot inside the splint with the foot flexed to provide the maximum angle of dorsiflexion.
3. Secure the calf straps with velcro closure.
4. Secure the forefoot strap with velcro closure.

