



Laundering and Care

1. Remove all panels and/ or any metal components prior to cleaning.
2. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
3. Outer shell may be wiped with mild soap and water
4. Do not use any substance containing Chlorine

Warning

1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This brace will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.

Caution

1. Single patient use only
2. Do not over-tighten closures
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.

EUROPEAN AUTHORIZED REPRESENTATIVE



Wellkang Ltd
Suite B, 29 Harley Street,
LONDON, W1G 9QR, U.K.
www.CE-Marking.EU

MANUFACTURED BY


Dicarre LLC

info@dicarre.com

Toll Free-1-844-DICARRE

125 Half Mile Road, Suite 200
Red Bank, NJ 07701

www.dicarre.com

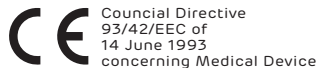


DICARRE

**Dicarre Night Splint With
Dorsiflexion Straps**

DA21

**User Guide
Application Instruction**

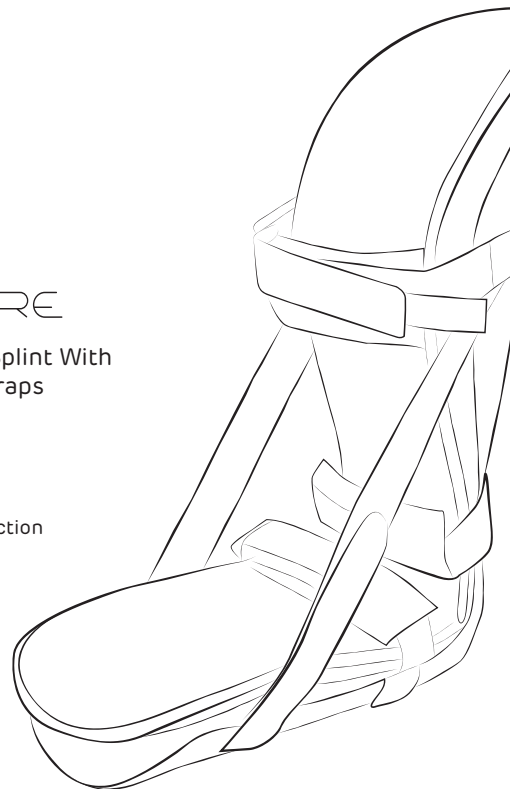

1. Undo all buckle closures and position affected foot inside the splint
2. Secure the ankle strap.(fig.1) Adjust the length of the contact closure strap until comfortable.
3. Secure the foot and calf straps.(fig.2,3) Adjust the length of the contact closure straps until comfortable
4. Adjust the dorsiflexion straps to desired foot angle and secure using fastener.(fig.4)

