



### Laundering and Care

1. Remove all panels and/or any metal components prior to cleaning.
2. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
3. Do not use any substance containing Chlorine

### Warning

1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This brace will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.

### Caution

1. Single patient use only
2. Do not over-tighten closures
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.

EUROPEAN AUTHORIZED REPRESENTATIVE



Wellkang Ltd  
Suite B, 29 Harley Street,  
LONDON, W1G 9QR, U.K.  
[www.CE-MarkIng.EU](http://www.CE-MarkIng.EU)



Council Directive  
93/42/EEC of  
14 June 1993  
concerning Medical Device

MANUFACTURED BY

**Dicarre LLC**

[info@dicarre.com](mailto:info@dicarre.com)

**Toll Free 1-844-DICARRE**

125 Half Mile Road, Suite 200  
Red Bank, NJ 07701


[www.dicarre.com](http://www.dicarre.com)

The logo for Dicarre Innovative Designs, featuring the word "DICARRE" in a bold, sans-serif font with a stylized "D".

DICARRE

Dicarre  
Sacro-Symphysis  
Belt

DB12  
User Guide  
Application Instruction

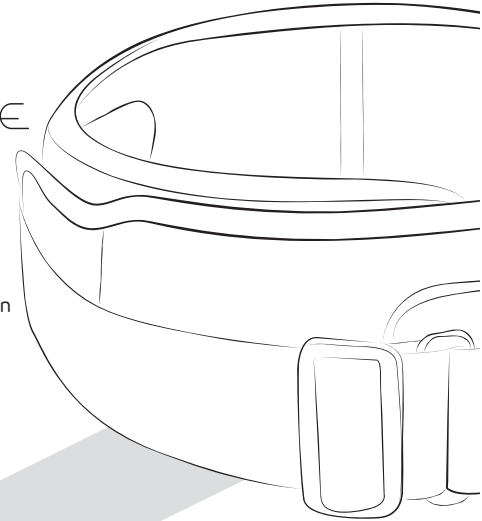

1. Place the brace around your hip.

2. Pull and wrap the sides around the right side overlapping the left with velcro closure.

3. Pull the webbing straps outward.

4. Pass the right strap through the D-ring of the left side and secure.

5. Readjust the length with the buckle of the left side if necessary and ensure that the brace fit is snug and does not cause any discomfort.

