



Laundering and Care

1. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
2. Do not use any substance containing Chlorine

Warning


1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This brace will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.

Caution


1. Single patient use only
2. Do not over-tighten closures
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.

DICARRE
Tennis Elbow Clasp

DE06
User Guide
Application Instruction

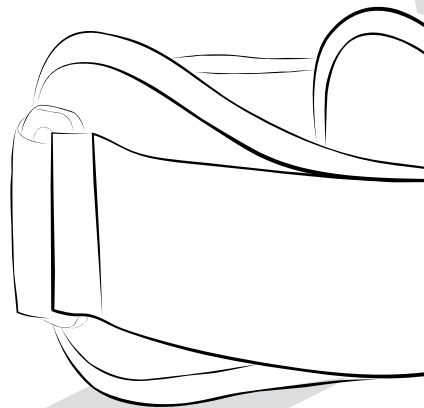

EUROPEAN AUTHORIZED REPRESENTATIVE

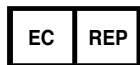

Wellkang Ltd
Suite B, 29 Harley Street,
LONDON, W1G 9QR, U.K.
www.CE-Marking.EU



Council Directive
93/42/EEC of
14 June 1993
concerning Medical Device

MANUFACTURED BY


Dicarre LLC

info@dicarre.com


Toll Free-1-844-DICARRE

125 Half Mile Road, Suite 200
Red Bank, NJ 07701


www.dicarre.com

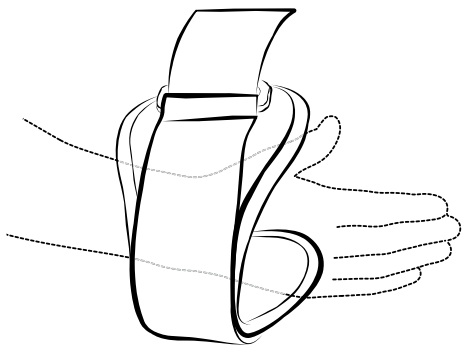


- 1 Placement**
Loosen the strap and slide hand into the elbow clasp

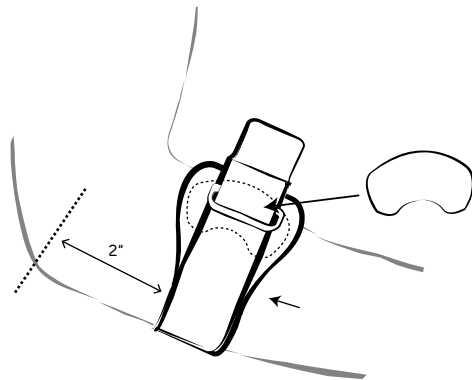

- 2 Position**
Place the tennis elbow clasp 2" inches below the lateral epicondyle (Prominent bone), inner pad should be placed beneath the elbow clasp on lateral side of the arm.

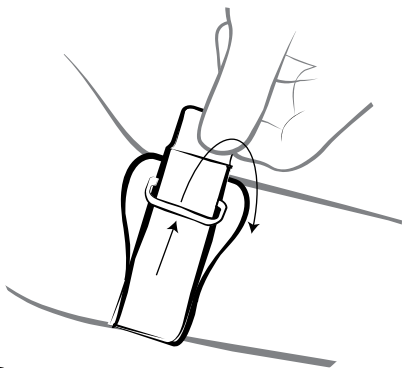

- 3 Secure strap**
Feed the strap through the D-ring and secure.

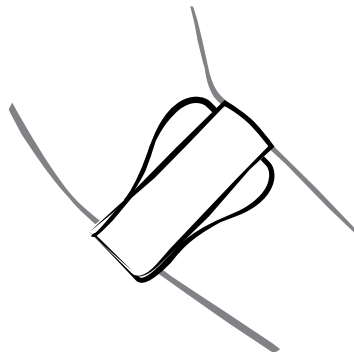

- 3 Fit adjustment**
Readjust the strap if necessary, ensure the brace is snug to desired tension.