



### **Laundering and Care**

1. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
2. Do not use any substance containing Chlorine

### **Warning**


1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This brace will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.

### **Caution**


1. Single patient use only
2. Do not over-tighten closures
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.

**DICARRE**  
Thumb Splint - Contour™

**DH04**  
User Guide  
Application Instruction

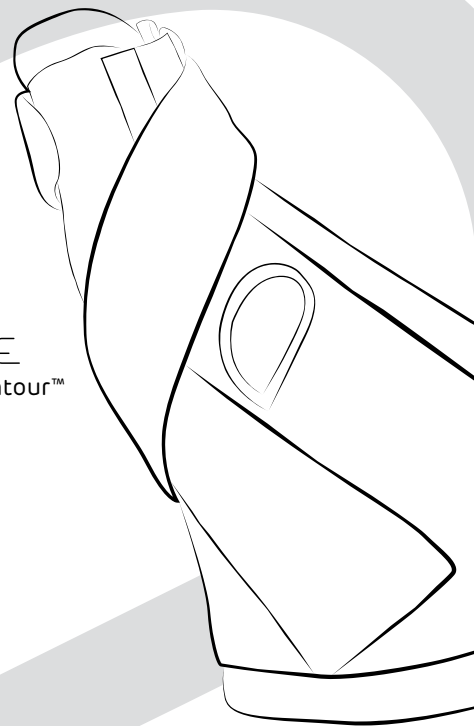

EUROPEAN AUTHORIZED REPRESENTATIVE

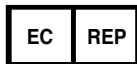

Wellkang Ltd  
Suite B, 29 Harley Street,  
LONDON, W1G 9QR, U.K.  
[www.CE-Marking.EU](http://www.CE-Marking.EU)

MANUFACTURED BY

**Dicarre LLC**

[info@dicarre.com](mailto:info@dicarre.com)

**Toll Free-1-844-DICARRE**

125 Half Mile Road, Suite 200  
Red Bank, NJ 07701

**[www.dicarre.com](http://www.dicarre.com)**



Council Directive  
93/42/EEC of  
14 June 1993  
concerning Medical Device

### 1. Placement

Slide affected hand through wrist band and into thumb orthosis. Make sure thumb and thumb strap at the front of the thumb splint are aligned.

### 2. Secure thumb strap

Feed the thumb strap through the D-ring and secure.

### 3. Secure wrist strap

Feed the wrist strap through the D-ring and secure.

### 4. Shape sewn-in metal stay

Press the D-shaped sewn pattern with unaffected hand to mold the inner metal stay for contoured fit

### 5. Figure 8 strapping

Pull & wrap elastic strap by figure-8 pattern as shown and secure.

### 6. Fit Adjustment

Readjust the strap to ensure firm support without impairing circulation.

