



Laundering and Care

1. Remove all panels and/or any metal components prior to cleaning.
2. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
3. Do not use any substance containing Chlorine


Warning

1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This brace will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.

Caution

1. Single patient use only
2. Do not over-tighten closures
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.

EUROPEAN AUTHORIZED REPRESENTATIVE

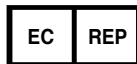

Wellkang Ltd
Suite B, 29 Harley Street,
LONDON, W1G 9QR, U.K.
www.CE-Marking.EU



Council Directive
93/42/EEC of
14 June 1993
concerning Medical Device

MANUFACTURED BY

Dicarre LLC

info@dicarre.com

Toll Free-1-844-DICARRE

125 Half Mile Road, Suite 200
Red Bank, NJ 07701


www.dicarre.com

DICARRE

**Dicarre Dynamic
Wrist-Hand-Finger
Orthosis**

DH09

**User Guide
Application Instruction**

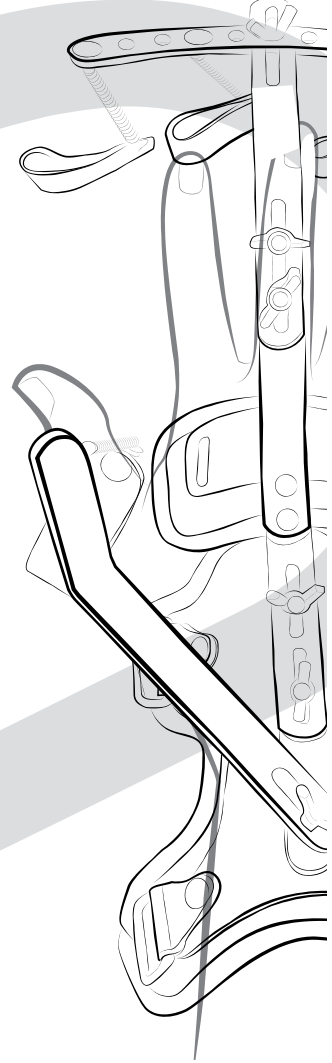

1. Assemble the finger control unit (A) palm and wrist (B) and thumb control unit (C) by the butterfly screws as shown in Fig 1
2. Turn the finger control unit to corresponding side according to left or right hand is applied. Fig 2
3. Adjust the length of the orthosis with the butterfly screw if necessary. Fig 3
4. Feed the velcro straps through the D-ring and secure. Fig 4
- 5 Feed the fingers through the leather loops. Fig 5
- 6 Adjust the degree of finger extension by adjusting the sections of finger control unit(B) with butterfly screws. Fig 6

