



### **Laundering and Care**

1. Remove all panels and/ or any metal frames or components prior to cleaning.
2. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
3. Do not use any substance containing Chlorine

### **Warning**

1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This support will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.
4. Settings of valgus/varus controls should be undertaken by medical professionals only

### **Caution**

1. Single patient use only
2. Do not over-tighten closures
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.

EUROPEAN AUTHORIZED REPRESENTATIVE



Wellkang Ltd  
Suite B, 29 Harley Street,  
LONDON, W1G 9QR, U.K.  
[www.CE-Marking.EU](http://www.CE-Marking.EU)



Council Directive  
93/42/EEC of  
14 June 1993  
concerning Medical Device

MANUFACTURED BY

**Dicarre LLC**

[info@dicarre.com](mailto:info@dicarre.com)

**Toll Free-1-844-DICARRE**

500 N. Rainbow Blvd., Suite 300,  
Las Vegas, NV 89107

**[www.dicarre.com](http://www.dicarre.com)**

**DICARRE**

**DK03**


**Three Panel Knee Immobilizer**

**DK05**

**Single Panel Knee Immobilizer**

**User Guide**

**Application Instruction**

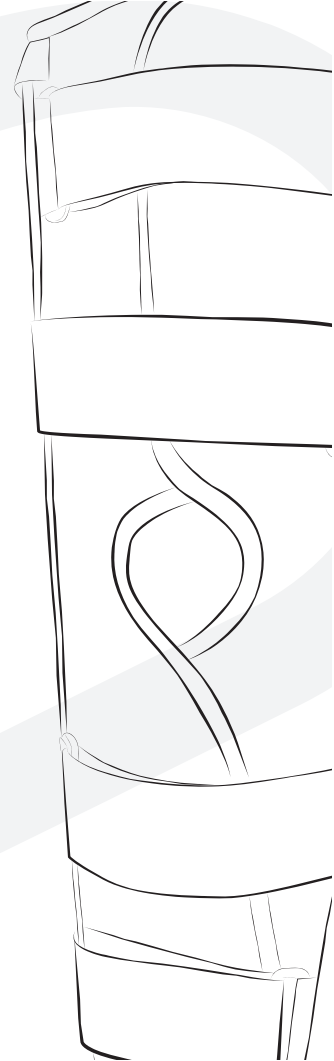


## Airmesh Pocket Sling

1. Undo all the contact closure straps
2. Position your knee on the knee immobilizer
3. The removable panels (DK03) may be adjusted laterally for correct sizing of thigh and calf circumference
4. Wrap and fasten the straps closest to the centre of the knee
5. Then proceed to fasten the upper thigh and calf straps.
6. Adjust the straps to a snug fit.

DK03

Dicarre Three Panel Knee Immobilizer

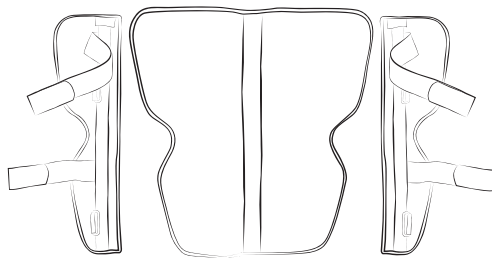

DK05

Dicarre Single Panel Knee Immobilizer

