



### Laundering and Care

1. Remove all plastic and/or any metal components prior to cleaning.
2. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
3. Do not use any substance containing Chlorine


### Warning

1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This brace will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.

### Caution

1. Single patient use only
2. Do not over-tighten closures
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.

EUROPEAN AUTHORIZED REPRESENTATIVE

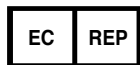

Wellkang Ltd  
Suite B, 29 Harley Street,  
LONDON, W1G 9QR, U.K.  
www.CE-Marking.EU



Council Directive  
93/42/EEC of  
14 June 1993  
concerning Medical Device

MANUFACTURED BY

**Dicarre LLC**

info@dicarre.com

**Toll Free-1-844-DICARRE**

125 Half Mile Road, Suite 200  
Red Bank, NJ 07701

**www.dicarre.com**


A detailed line drawing of a hinged knee wrap, showing the various straps, buckles, and the central hinge mechanism. The drawing is positioned on the right side of the page, partially overlapping a large, light gray curved graphic element.

**DICARRE**  
Hinged Knee Wrap

**DK16**  
User Guide  
Application Instruction

## 1. Placement and wrap closure

The brace is put on, with the center of the hinges (Fig. 1) to the joint line (both medial & lateral). Then, wrap around with the velcro closure. (Fig. 2)

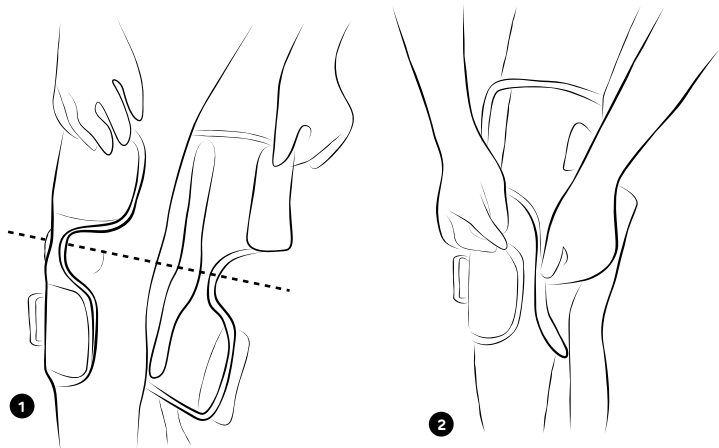

## 2. Secure straps

Starting with the upper strap, feed straps through the D-rings and secure. (fig 3, 4) Fit should be snug but not too tight as to restrict circulation.

