



Laundering and Care

1. This mitts can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
2. Do not use any substance containing Chlorine


Warning

1. Carefully read this application instruction and warning prior to fitting mitts, failure to properly position the mitt and fasten each strap will compromise performance and support.
2. DO NOT USE if patient IV site or wound site could be compromised by the mitt
3. DO NOT USE if affected limb is fractured or dislocated
4. DO NOT USE if a patient is highly aggressive, agitated or suicidal

Caution

1. If the applied device severely limits movement or is uncomfortable, this could lead to severe emotional, psychological and physical problems. If such symptoms appear or worsen, it is recommended to reassess use of this product for this patient
2. Ensure mitts are properly secured
3. Ensure mitts are not torn or damaged
4. Check patient cannot use his/her teeth to remove mitts
5. Do not allow patient to ingest mitt material
6. Be aware patients wearing mitts whilst walking could be at risk of falling and should be closely monitored

EUROPEAN AUTHORIZED REPRESENTATIVE

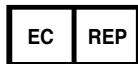

Wellkang Ltd
Suite B, 29 Harley Street,
LONDON, W1G 9QR, U.K.
www.CE-Marking.EU



Council Directive
93/42/EEC of
14 June 1993
concerning Medical Device

MANUFACTURED BY

Dicarre LLC

info@dicarre.com

Toll Free-1-844-DICARRE

125 Half Mile Road, Suite 200
Red Bank, NJ 07701

www.dicarre.com

DICARRE


Patient Safeguard Mitts

DM06a, DM06b

DM07a, DM07b

User Guide


Application Instruction

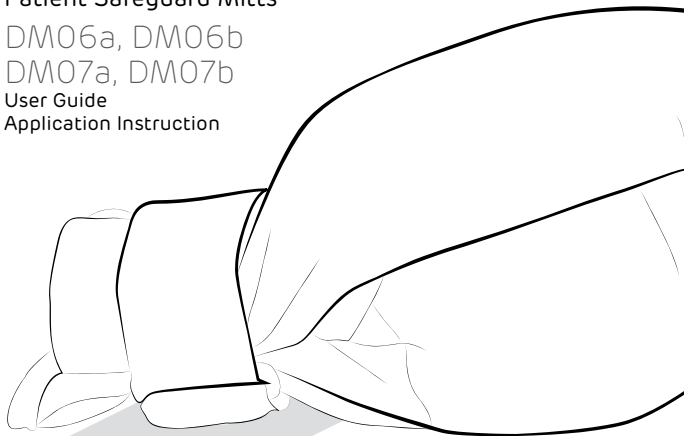

DM06a

Dicarre Double Padded Peep Mitts


1. Place patient's hand into the mitt with the palm facing downwards. (When the inspection flap is lifted open the dorsal surface of the hand should be visible).
2. Secure the wrist strap around the patient's wrist at the narrowest part. Feed the strap through the D-ring and fasten the strap with velcro closure.
3. In order to ensure that circulation is not compromised, it should be possible to slide one finger between the strap and the patients wrist.
4. To use hospital bed connection straps - wrap the strap around the patients wrist or through the loops on the mitt and around an appropriate area of the bed frame. Secure with buckle as required.

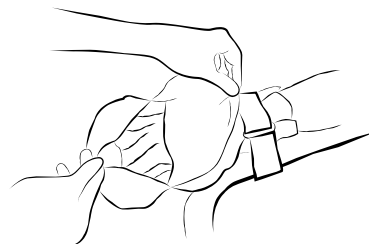

DM06b

Dicarre Double Padded Security Mitts

1. Place patient's hand into the mitt with the palm facing downwards.
2. Secure the wrist strap around the patient's wrist at the narrowest part. Feed the strap through the D-ring and fasten the strap with velcro closure.
3. In order to ensure that circulation is not compromised, it should be possible to slide one finger between the strap and the patients wrist.
4. To use hospital bed connection straps - wrap the strap around the patients wrist or through the loops on the mitt and around an appropriate area of the bed frame. Secure with buckle as required.

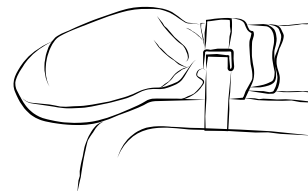

DM07a

Dicarre Coolmesh Security Mitts

1. Place patient's hand into the mitt with the mesh on the dorsal surface.
2. Secure the wrist strap around the patient's wrist at the narrowest part. Feed the strap through the D-ring and fasten the strap with velcro closure.
3. In order to ensure that circulation is not compromised, it should be possible to slide one finger between the strap and the patients wrist.
4. To use hospital bed connection straps - wrap the strap around the patients wrist or through the loops on the mitt and around an appropriate area of the bed frame. Secure with buckle as required.



DM07b

Dicarre Coolmesh Finger Control Mitts

1. Place patient's hand into the mitt with the mesh on the dorsal surface.
2. Secure the wrist strap around the patient's wrist at the narrowest part. Feed the strap through the D-ring and fasten the strap with velcro closure.
3. In order to ensure that circulation is not compromised, it should be possible to slide one finger between the strap and the patients wrist.
4. To use hospital bed connection straps - wrap the strap around the patients wrist or through the loops on the mitt and around an appropriate area of the bed frame. Secure with buckle as required.

