



### **Laundering and Care**

1. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
2. Do not use any substance containing Chlorine

### **Warning**

1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This brace will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.

### **Caution**

1. Single patient use only
2. Do not over-tighten closures
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.

EUROPEAN AUTHORIZED REPRESENTATIVE



Wellkang Ltd  
Suite B, 29 Harley Street,  
LONDON, W1G 9QR, U.K.  
[www.CE-Marking.EU](http://www.CE-Marking.EU)



Council Directive  
93/42/EEC of  
14 June 1993  
concerning Medical Device

MANUFACTURED BY

**Dicarre LLC**

[info@dicarre.com](mailto:info@dicarre.com)

**Toll Free 1-844-DICARRE**

125 Half Mile Road, Suite 200  
Red Bank, NJ 07701

**[www.dicarre.com](http://www.dicarre.com)**

A line drawing of the Dicarre Phoenix Tracheo Collar, showing the front view of the device with its straps and buckles.

**DICARRE**


Dicarre Phoenix Tracheo Collar

**DN03**


User Guide

Application Instruction


- 1 Undo all contact closure straps.

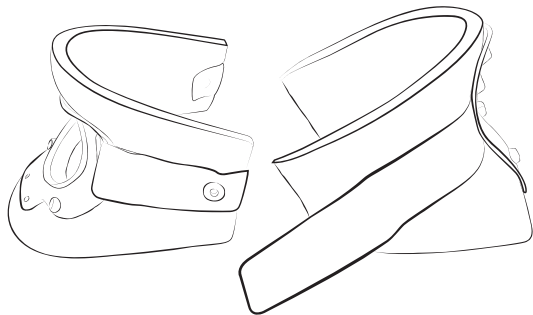

- 2 After selecting the properly sized Collar, apply the BACK piece of the Collar to the back of the patient's neck. Center the Collar.

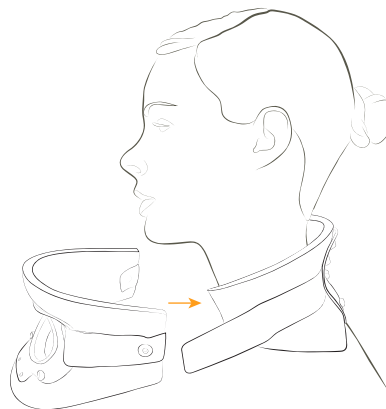

- 3 Apply the FRONT piece of the Collar with the chin secured in the recess. Center the Collar and ensure neutral alignment. The FRONT piece OVERLAPS the BACK piece to ensure effective immobilization and comfort.

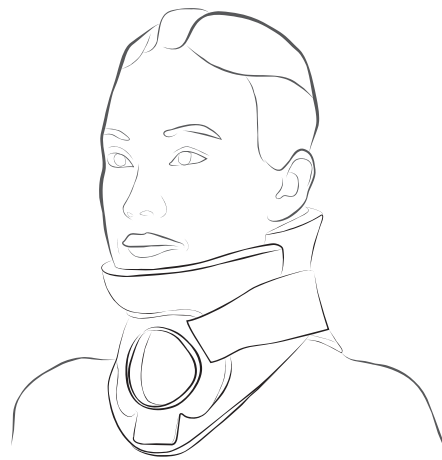

- 4 With Velcro hook and loop fasteners, tighten the Collar with a bilateral adjustment. This will secure the patient's cervical region in neutral alignment.