



#### **Laundering and Care**

1. Remove all panels and/ or any metal frames or components prior to cleaning.
2. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
3. Do not use any substance containing Chlorine

#### **Warning**

1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This support will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.


#### **Caution**

1. Single patient use only
2. Do not over-tighten closures
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.

EUROPEAN AUTHORIZED REPRESENTATIVE



Wellkang Ltd  
Suite B, 29 Harley Street,  
LONDON, W1G 9QR, U.K.  
[www.CE-Marking.EU](http://www.CE-Marking.EU)

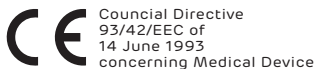

MANUFACTURED BY

**Dicarre LLC**

[info@dicarre.com](mailto:info@dicarre.com)

**Toll Free-1-844-DICARRE**

125 Half Mile Road, Suite 200  
Red Bank, NJ 07701


**[www.dicarre.com](http://www.dicarre.com)**



**DICARRE**  
**Dicarre Tracheo TX Collar**  
**DN19**


**Dicarre Tracheo Collar**  
**DN09**

User Guide  
Application Instruction

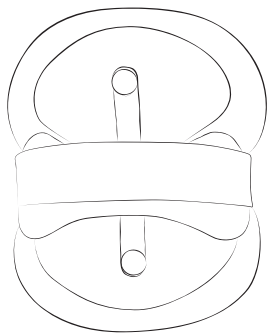



**DN19**

Dicarre Tracheo TX Collar

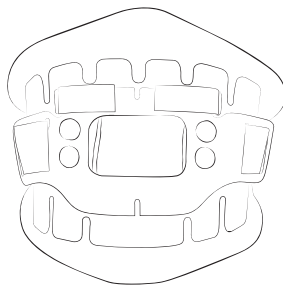

**DN09**

Dicarre Tracheo Collar

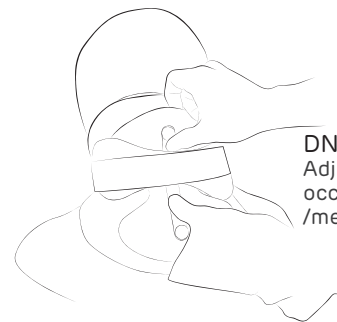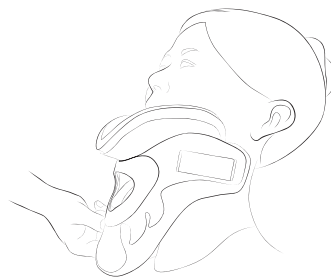

**DN19**  
Adjust Malleable  
occipital fibre-plastic  
/metal stay

1. Place the head in the desired position. Measure the length from the chin to the sternum.
2. Roll anterior and posterior panel
3. Place chin piece of anterior section directly under the chin
4. After maintaining desired position, attach posterior closure straps to anterior sides
5. Depending on if patient is fitted in supine or ambulatory position, straps may need adjustment accordingly.