



Laundering and Care

1. Remove all panels and/or any metal components prior to cleaning.
2. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
3. Do not use any substance containing Chlorine


Warning

1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This brace will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.

Caution

1. Single patient use only
2. Do not over-tighten closures
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.

EUROPEAN AUTHORIZED REPRESENTATIVE

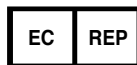

Wellkang Ltd
Suite B, 29 Harley Street,
LONDON, W1G 9QR, U.K.
www.CE-Marking.EU



Council Directive
93/42/EEC of
14 June 1993
concerning Medical Device

MANUFACTURED BY

Dicarre LLC

info@dicarre.com


Toll Free-1-844-DICARRE


125 Half Mile Road, Suite 200
Red Bank, NJ 07701

www.dicarre.com

DICARRE
Dicarre Wrist Brace &
Dicarre Wrist / Thumb Brace


DW03, DW04
User Guide
Application Instruction

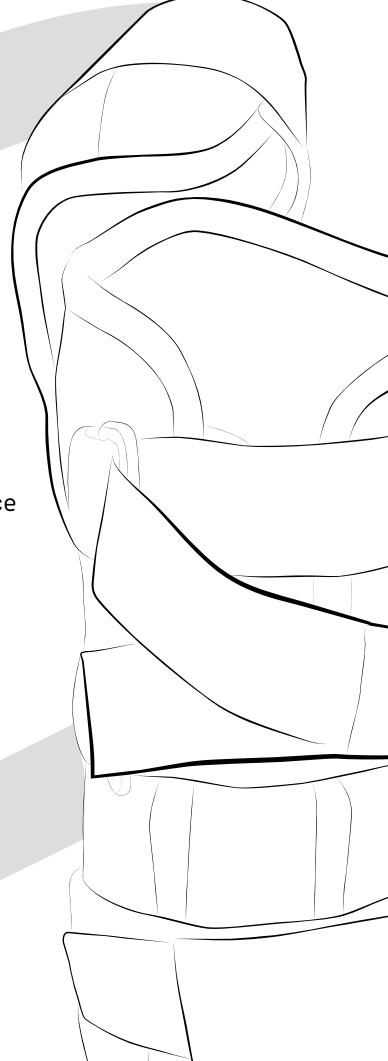

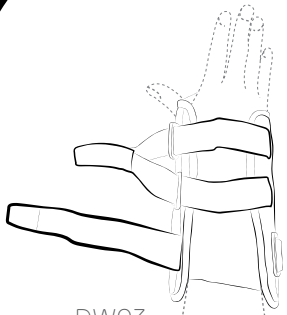


DW03
Dicarre Wrist Brace

- 1 Placement**
Undo all contact closure straps. Slide affected hand into wrist brace

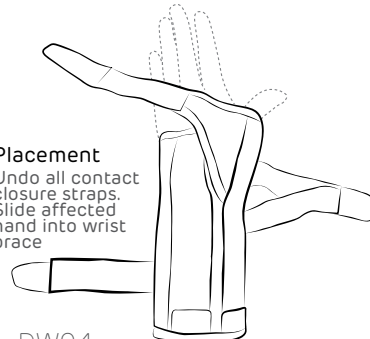

DW04
Dicarre Wrist / Thumb Brace

- 2 Secure straps**
Starting with the upper wrist strap, wrist and secure by velcro closure. (DW04) Wrap the thumb strap and secure

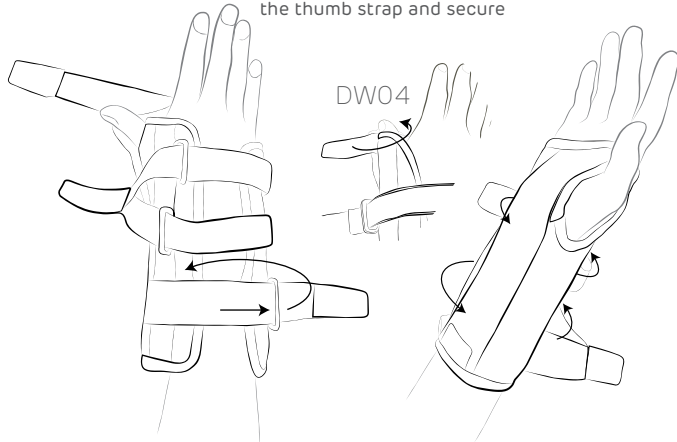

DW03
Dicarre Wrist Brace

- 4 Fit adjustment**
Fasten all contact closure straps and secure to a snug fit. Adjustments may be necessary as hand, wrist and thumb swelling patterns change.



DW04
Dicarre Wrist / Thumb Brace