



Laundering and Care

1. Remove all panels and/ or any metal components prior to cleaning.
2. This brace can be hand washed in cold or lukewarm water with mild soap or detergent and air dried
3. Outer shell may be wiped with mild soap and water
4. Do not use any substance containing Chlorine

Warning

1. Carefully read this application instruction and warning prior to fitting braces, failure to properly position the brace and fasten each strap will compromise performance and support.
2. This brace will Not prevent or reduce all injuries. Proper rehabilitation and activity modification are essential while wearing the brace.
3. Federal Law restricts the sale of this Device to anyone other than a Licensed Health Care Provider.

Caution

1. Single patient use only
2. Do not over-tighten closures
3. If irritation, pain or discomfort occurs while wearing this brace, discontinue use and immediately consult your medical professional.

EUROPEAN AUTHORIZED REPRESENTATIVE



Wellkang Ltd
Suite B, 29 Harley Street,
LONDON, W1G 9QR, U.K.
www.CE-Marking.EU

MANUFACTURED BY

Dicarre LLC

info@dicarre.com

Toll Free-1-844-DICARRE

125 Half Mile Road, Suite 200
Red Bank, NJ 07701


www.dicarre.com

DICARRE

Dicarre Podus Boot


DA20

User Guide
Application Instruction

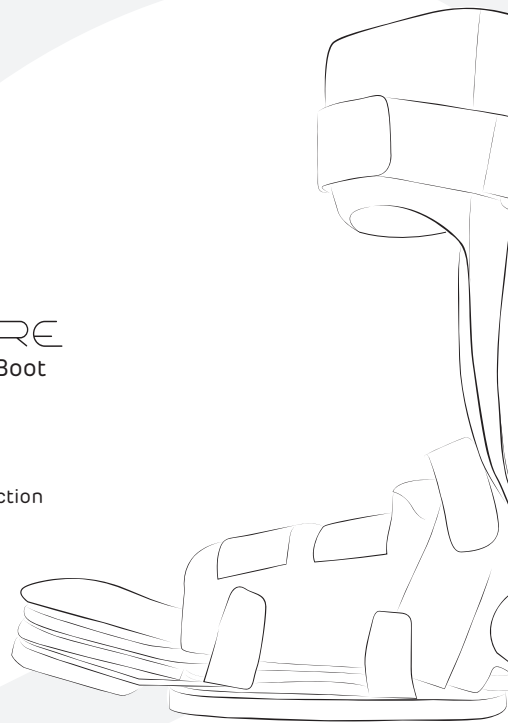


Council Directive
93/42/EEC of
14 June 1993
concerning Medical Device


1. Open all hook and loop closures. Flex knee to approximately 45° and place foot into boot. Make sure sole of foot is resting on bottom of boot, and calf of leg is resting on the calf portion of boot (fig.1).
2. Wrap left side of foot cuff over instep. Wrap right side of foot cuff over top of left and secure velcro closure (fig. 2).
3. Wrap left side of calf. Wrap right side of calf cuff over top of left and secure velcro closure (fig 3).
4. Wrap and secure the calf strap by velcro closure (fig 4).

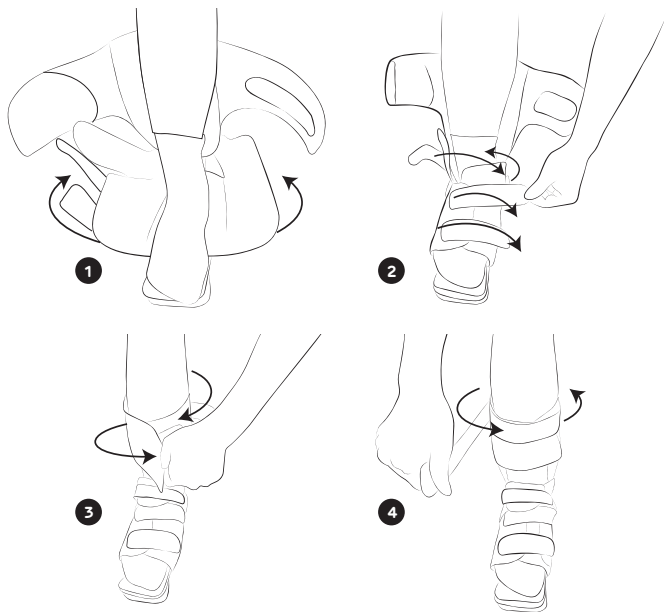

Rotation stabilizer bar - Helps control external rotation

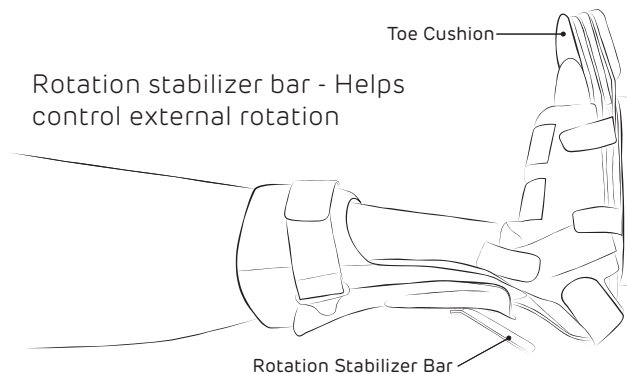

Length adjustment

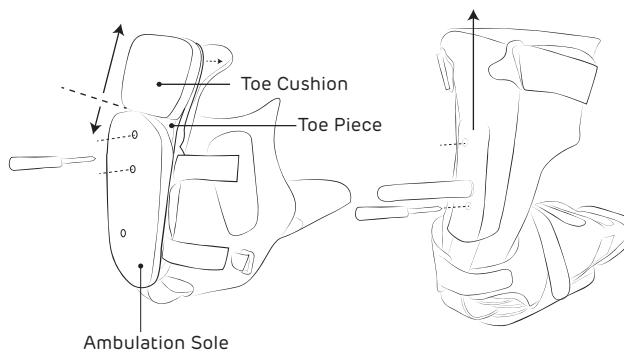

Note: Ensure heel does not come into contact with boot frame. Heel should "float" in the boot.